

# PMU<sup>®</sup>power

pen precision



## USER MANUAL

PLEASE READ THE MANUAL BEFORE USE





① Battery

② Wired connector

③ Charger

## Wireless Mode

The instrument working with rechargeable battery

## Wired Mode

The instrument working with alternating current




### Charging Status

The power indicator stays red while charging

The power indicator turns green after charging completed

## Preparation

1. Take out needle cartridge in the sterilized packaging
2. Fitting round slot 1 into 2 of machine, push needle cartridge horizontally up to the end.
3. Lock the needle cartridge by turning it to the counterclockwise
4. Turn the needle cartridge clockwise and then push out horizontally if you want to remove it.

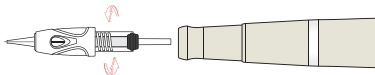

5. Start the **PMU**power pen precision

Press the power button for 2-3 seconds to start or shut down the device.


## Attention

In order to protect the instrument well, do not invert the instrument to prevent the liquid backflow

## Adjustment of needle length

Adjust the depth of needle by turning the scale adjustment ring

Turn it clockwise, needle depth will be shorter Turn it counterclockwise, needle depth will be longer

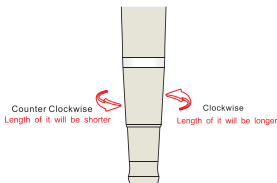

## Speed Display



## Speed Adjustment

The speed after start the device is the same as the speed before shutdown

The speed changes once each time you press the button

# Needle cartridge introduction

**PMU**power<sup>®</sup>  
pen precision



L1



M1



R3-M



R5

**L1:** 1-PIXEL LINER HYGIENE MODULE, Number: PMU-L1-S

**M1:** 1-NANO LINER HYGIENE MODULE, Number: PMU-M1-S

**R3-M:** 3-NANO LINER HYGIENE MODULE, Number: PMU-R3-M-S

**R5:** 5-NANO LINER HYGIENE MODULE, Number: PMU R5-S

L1	M1	R3-M	R5
Pixel Brows	Lashliner	Soft Eyeliner	Areola
Powder Brows	Hairstrokes (Dry skin)	Ombre Brows	Scars
Eyeliner	Soft Ipliner	Soft Ipliner	Lipstick Effect
Shaded Eyeliner	Areola	Powder Lips	Remover
Hair Strokes (Oily skin)		Areola	
Pixel Lips			



**Model name: PMUpower pen precision**

Adapter: 5V-1000mA Package Weight: 425g

Package Size: 169\*123\*56 mm

## Packing List

Item	Quantity
Hand Piece	1 pc
Adapter	1 pc
Manual	1 pc
Power Cable	1 pc
Charger	1 pc
Batteries	2 pcs
Needle Cartridge	40 pcs
Pen Holder	1 pc
Portable Storage Bag	1 pcs

	Please be sure to read instructions before use and follow as described.
	<b>Do not apply it to the man as follow:</b> 1. Who has acute disease or bloody disease over 38°C 2. Who has allergy and hemophilia 3. Who has melanosis of the face 4. Who has high internal heat 5. Who regularly visits hospital 6. Who takes medicines 7. Who is under 12 years old
	Do not use it on the power other than AC 220V. It may cause fires or malfunction
	Do not use it near the place with high temperature, dust or humidity (Near the humidifier). It may cause malfunction.
	Do not repair, disassemble or alter the controller and the body. It may cause malfunction.
	Do not touch the device or detach the power cords with wet hands. It may cause electric shock.
	Do not use the stripped cord or propped cord. It may cause fires (Be sure to replace it with another).
	Do not use the propped cord which has crack or scars. It may cause scars on the skin.
	When you don't use it for a long time, unplug it from the outlet.
	Be sure to have an inspection for test. If any troubles are found, get it repaired.
	No one other than the operator can touch it. It may cause accidents or malfunction.

	220 V		
	Direct current		PMU-line ApS
	Class I equipment		0865
	Attention, consult accompanying documents		PMU-line ApS
	2/2023		Floating applied part



The warranty period of **PMUpower pen precision** is 12 months.

We offer repair to our products from the day they are purchased. If any of our products has damages or natural trouble. We offer repair or replacements,

## **General maintenance**

Make sure to inspect the device and adapter before use

Keep it with its power off. Wipe its surface out with a dry cloth

Keep it away from direct sunlight or high temperature and humidity

Do not leave the measurement, repair or alter of the device to non-professional staff

When you use the device which has not been used for a long time, make sure to check if it works properly with safety

**PMU**power®  
pen precision

**PMU**-line®  
Permanent Makeup Products

Juelsmindevej 57, DK-7120 Vejle, Denmark- [www.pmu-line.com](http://www.pmu-line.com)

**Designed by PMU-line in Denmark**  
Assembled in P.R.C.